



iTero Scanner Instructions (Two Approaches)

Pre-iTero Scan Instruction:

1. Scan as an iRecord.
2. Do not choose to send to a lab.
3. After you scan your patient's mouth, the scan can be found immediately on MyiTero.com or between approximately fifteen minutes to two days on MyAligntech.com.

Option 1: MyiTero.com Export to STL File Instruction:

1. Log into MyiTero.com.
2. Enter the patients name into the search field (if you can't find it, make sure you are choosing the correct scanner location).
3. Click on the patient's name.
4. Click "Export".
5. An export pop-up box will open and you will see these drop down questions:
 - a. **Export type:** Choose "Solid model with low profile base"
 - b. **Data format:** Choose "Files per arch (arches oriented in occlusion)"
 - c. **File type:** Choose "STL, no color"
 - d. Click "Export".

Note: The STL file is large and may take a minute or two to download. If the export is taking too long, check to make sure that your pop-up blocker is **off** for this site.
6. A window will pop up containing several files associated with your scan. You can copy and paste, drag and drop, or select all, and then save these files to your patient's file.


Option 2: MyAligntech.com Export to STL File Instruction:

1. Log into MyAligntech.com.
2. Click on the ID number for your patient.
3. Click "Export (OrthoCAD 3.5 or higher)".
4. OrthoCAD export pop-up box will open and you will see these drop down questions:
 - a. **Export type:** Choose "Solid model with low profile base"
 - b. **Data format:** Choose "Files per arch (arches oriented in occlusion)"
 - c. **Folder name:** Enter your patient's name in the same format as your chart.
 - d. **File type:** Choose "STL, no Color"
 - e. Click "Export".
5. A dialogue box will pop up asking if you would like to open the exported file's location. Choose "Yes".
6. A window will pop up containing several files associated with your scan. You can copy and paste, drag and drop, or select all, and then save these files to your patient's file.



How to attach files to your patient case once the scans have been exported to your desktop:

1. Login to your DDX Account

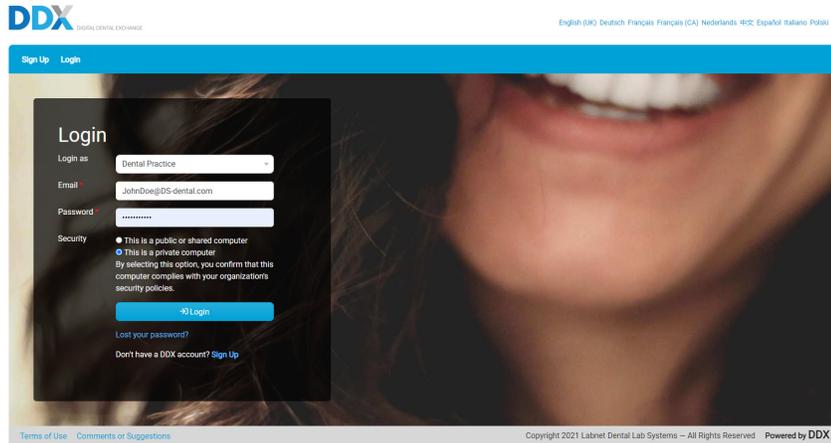

2. Select New Case



3. Enter all required patient information and details in the work request form

Portal SLX Clear Aligner Lab - Cases - Finances - New Task - Search

Schedule a New Case

Patient Information

Provider * Dr. John Doe Gender

First Name Chart Number * 12345 Test Case

Last Name Birth date mm/dd/yyyy

Work Requested

SLX Clear Aligners
Standard, Lite, or Express Package
Remove Procedure

Primary Concern
Chief Complaint *

Case Properties
Impression Type * Treatment Steps * Lab to Recommend *

Arch Development
Archform Width * Optimize * Midline * Optimize *

A-P Relationship

A-P relationship: Right Canine Optimize Class I * Overjet / Underbite * Optimize *

A-P relationship: Left Canine Optimize Class I * Overbite * Optimize *

A-P relationship: Right Molar Maintain * Crossbite * Optimize *

A-P relationship: Left Molar * Maintain *



4. Under Case Files, select the upload button to add scan files from your desktop folder

Enclosures

Case Files **Upload** 12 Specified Files Recommended

No files uploaded

Notes

Subject: [text] Apply To: [text]

Send Date: 05/06/2021


Type: Emergency

Tags: [dropdown]

Coupon Code: [dropdown]

Case for SLX Aligner Aligner Lab Uploads

Select Files to Add

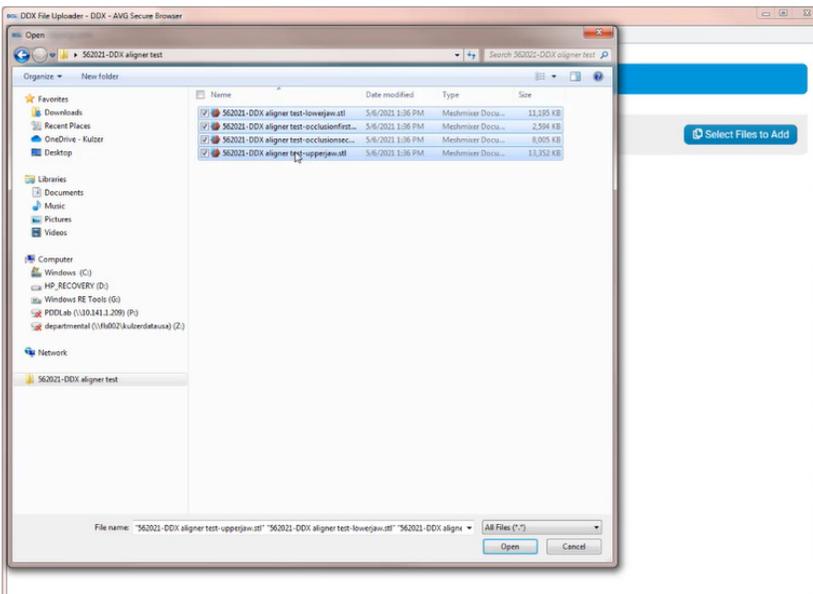

Case for SLX Aligner Aligner Lab Uploads

 562021-DDX aligner test-lowerjaw.stl 100% File Uploaded	Patient: Unknown	Specified File: [dropdown]
 562021-DDX aligner test-occlusionfirst.stl 100% File Uploaded	Patient: Unknown	Specified File: [dropdown]
 562021-DDX aligner test-occlusionsecond.stl 100% File Uploaded	Patient: Unknown	Specified File: [dropdown]
 562021-DDX aligner test-upperjaw.stl 87.92% Uploading	Patient: Unknown	Specified File: [dropdown]

Select Files to Add



5. The uploaded files will appear as thumbnails under the Case Files section. Enter the notes relevant to your patient, check to accept terms & conditions before submitting your aligner case.

Case Files 12 Specified Files Recommended

562021-DDX aligner test-lowerjaw_1.stl
562021-DDX aligner test-occlusionfirst_1.stl
562021-DDX aligner test-occlusionsecond_1.stl
562021-DDX aligner test-upperjaw_1.stl

Notes

Send Date: 05/06/2021
Tags: Try-in Emergency
Coupon Code:

1000 characters remaining

I accept all Terms and Conditions

Submit Case Save Draft

Terms of Use Comments or Suggestions Copyright 2021 Labnet Dental Lab Systems - All Rights Reserved Powered by DDX