




3Shape Digital Integration Guide

Step 1. Login to 3Shape Communicate, click on the “**Connections**” tab and search for your Aligner Lab via the labs main contact email or name.

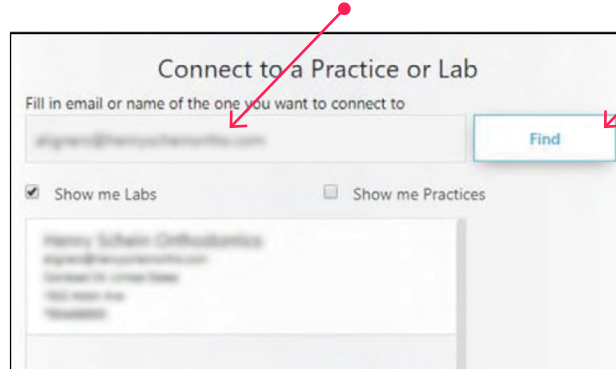

Step 2. Under Settings, choose “**Lab Connections**” and select “**Update**”. The Aligner Lab information will appear as a lab on the office’s TRIOS.

For the following global Aligner Labs, please confirm with your representative which Lab to connect with.

Use the following emails to make your connection:

Target SLX Aligner Lab (Italy): aligner@targetortodonzia.it

NAOL Australia SLX Aligners: slxaligners@naol.com.au

Organizacion Ortho SLX Aligners (Mexico): slx@organizacionortho.com.mx

Ortho-Trends SLX Aligners (Belgium): info@orthotrends.com

HSO Intl Portal: alignersintl@henryschein.com


Orto Line SLX Ukraine: website@ortoline.com.ua

Cerum Canada: info@cerumorthoaligners.com


KTT SLX Japan: slxclearaligners@ortho.co.jp

Step 3. Logout.

Step 4. Login to **DDX**, click on the drop-down arrow on the top right hand side of the page, to choose **Store**.

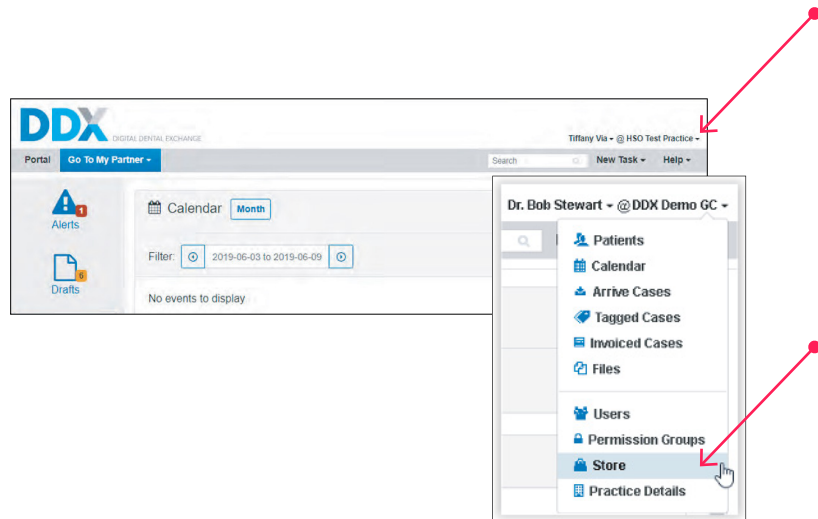

Step 5. Scroll down and find TRIOS 3Shape Communicate and click on “Enable”.

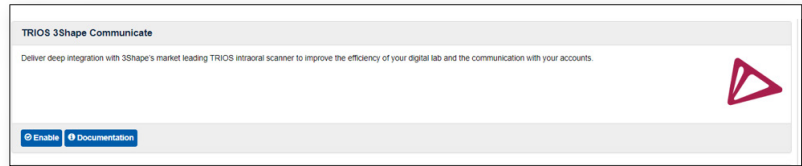


Step 6. Click on “Configure your add-on”.



Step 7. Click on “Authorize” and type in the email and password of your Communicate account.



Step 8. Now the two networks are connected! Before performing a TRIOS scan, select your **SLX Aligner Lab** as the receiver and DDX will create a Draft case.

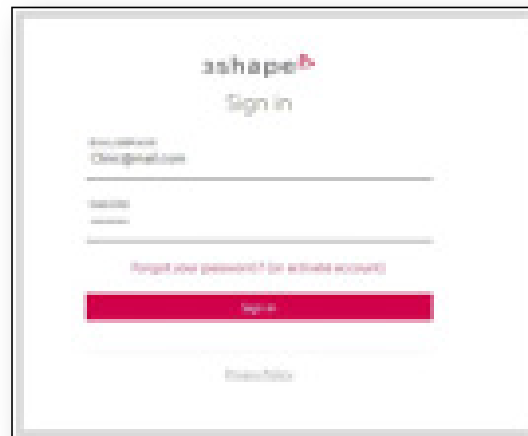

SLXClearAligners.com | Tel: +(1) 760-448-8600

Ortho Organizers, Inc. 1822 Aston Ave., Carlsbad, CA 92008-7306
© 2021 Ortho Organizers, Inc. All rights reserved. M2311

