




# VivaScan .STL Export Workflow Steps

**Instructions:**

1. Go to the "Send" tab
2. Verify the .STL file is checked (uncheck .PLY file)
3. Uncheck "Anonymous Patient"
4. Click on the "Save to File" button
5. Create a folder and save files to be added to DDX

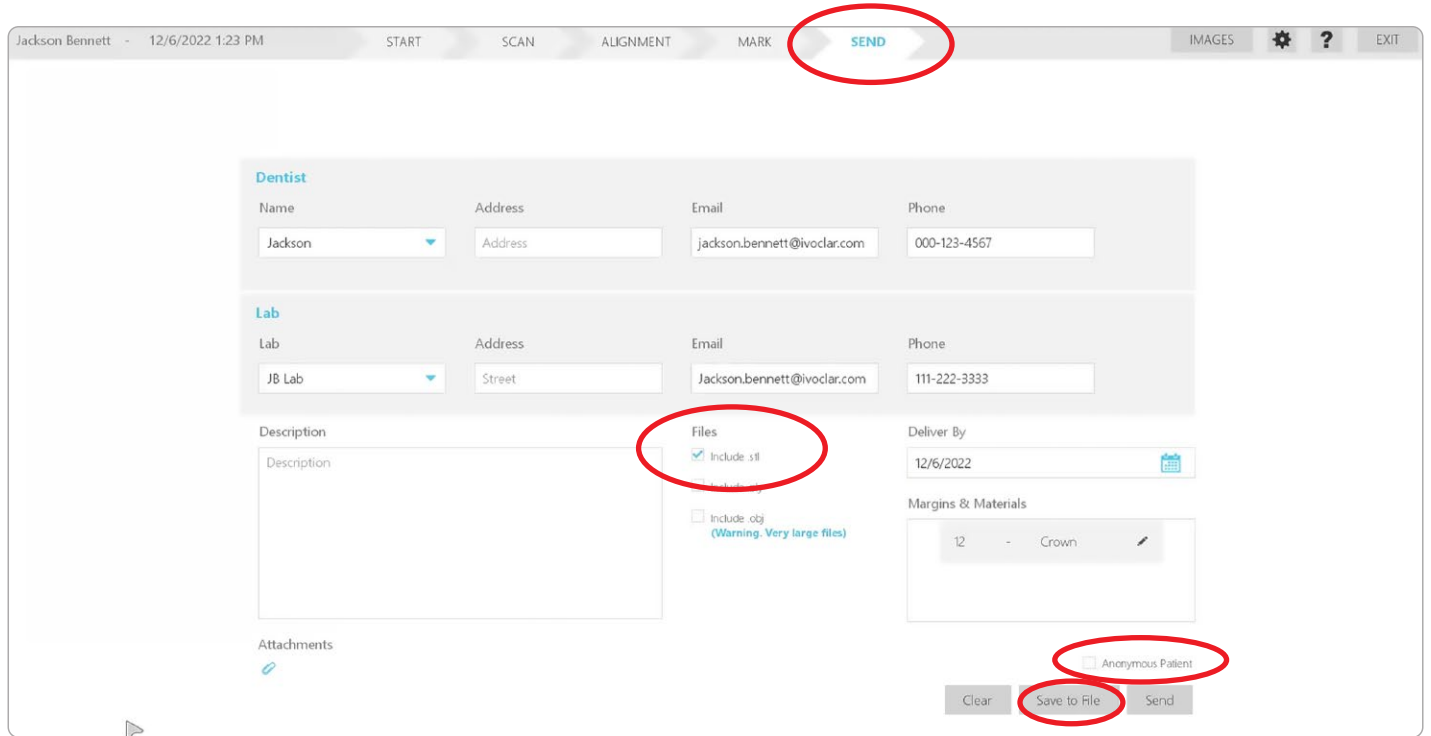

**Note:** Depending on what was scanned, you will find multiple .STL files in the directory. If you have completed two bite scans, you will have the corresponding .STL files for each bite.

Name	Type
F23-04F_Case_instructions	Text Document
F23-04F_Case_manifest	Text Document
F23-04F_leftbite_buccal	STL File
F23-04F_lower_lower	STL File
F23-04F_rightbite_buccal	STL File
F23-04F_upper_upper	STL File

## 6. Login to your DDX Account

DDX DIGITAL DENTAL EXCHANGE

English (UK) Deutsch Français Français (CA) Nederlands 中文 Español Italiano Polski 日本

Sign Up Login

### Login

Login as: Dental Practice

Email: [input field]

Password: [input field]

Security:

- This is a public or shared computer
- This is a private computer

[button: Login]

Lost your password?

Don't have a DDX account? Sign Up

## 7. Select new case

DDX DIGITAL DENTAL EXCHANGE

Invite a Colleague Help Demo HSO Sales @ HSO Sales Demo

Portal Go To My Partner

New Task Search

My Portal

Alerts 3 Alerts 3

Communications

SLX Aligner Lab

New Case

Sales Demo - Aligners News & Offers

New Case

## 8. Enter all required Patient Information and details in Work Requested

Portal SLX Aligner Lab Cases Finances New Task Search

### Schedule a New Case

#### Patient Information

Provider: Dr. Test Acct Gender: [input field]

First Name: [input field] Chart Number: [input field]

Last Name: [input field] Birth date: mm/dd/yyyy [input field]

#### Work Requested

SLX Clear Aligners Standard, Lite, or Express Packages [button: Remove Procedure]

Primary Concern: Chief Complaint: [input field]

Case Properties: Impression Type: PVS Impression (\$19.99) Arches To Treat: Both Arches Treatment Steps: Lab to Recommend

Arch Development: Archform Width: Maintain Midline: Improve with IPR -0.3 - 0.4

9. Under Case Files, select the Upload button to add scan files

Enclosures

Case Files **Upload**

12 Specified Files Recommended

No files uploaded

Notes

Subject: [dropdown] Apply Te... [dropdown]

Send Date: 06/16/2022

Try-in Emergency

Tags: [dropdown]

Coupon Code: [dropdown]

800 characters remaining.

I accept all Terms and Conditions

Submit Case Save Draft

Case for SLX Aligner Lab Uploads

Select Files to Add

Case for SLX Aligner Lab Uploads

Open

File name: "47873979\_iprofile\_occlusion\_u" \*478? All Files

Open Cancel

Case for SLX Aligner Lab Uploads

27449669\_shell\_lockedocclusion\_ul.stl  
100%  
File Uploaded Clear

27449669\_shell\_occlusion\_l.stl  
100%  
File Uploaded Clear

27449669\_shell\_occlusion\_l.stl  
100%  
File Uploaded Clear

27449669\_shell\_occlusion\_u.stl  
100%  
File Uploaded Clear

IMG0001.JPG  
100%  
File Uploaded Clear

10. The uploaded files will appear as thumbnails under the Case Files section. Enter the notes relevant to your patient, and accept the Terms & Conditions before submitting your aligner case

The screenshot shows a software interface for submitting a case. At the top, there is a 'Case Files' section with an 'Upload' button and a notification for '12 Specified Files Recommended'. Below this, five file thumbnails are displayed, each with a tooth icon and a filename: '47873979\_lprofile\_occlusion\_l.stl', '47873979\_lprofile\_occlusion\_u.stl', '47940121\_lprofile\_occlusion\_l.stl', '47940121\_lprofile\_occlusion\_u.stl', and '51782637\_lprofile\_occlusion\_l.stl'. A red oval highlights this entire 'Case Files' section.

Below the files is a 'Notes' section. It features a rich text editor with a toolbar containing icons for bold, italic, underline, bulleted list, numbered list, link, unlink, and image. To the right of the editor are fields for 'Send Date' (set to 12/15/2022), 'Tags' (with a 'Try-in' button and an 'Emergency' button), and a 'Coupon Code' field. At the bottom of the notes section, there is a checkbox labeled 'I accept all Terms and Conditions' and two buttons: 'Submit Case' and 'Save Draft'. A red oval highlights this checkbox and the buttons.

800 characters remaining.